Prior to use, carefully read the instructions for use.

GC Tooth Mousse

TOPICAL CREME WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE
For use by or under the supervision of a dental professional in the recommended indications.

Description
GC Tooth Mousse is a water based creme containing RECALDENT™ (CPP-ACP: Kazein Fosforanetat – Амфориан Кальция Фосфат). Korogo CPP-ACP (Casein Phosphopeptide – Амфориан Кальций Фосфат). When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bioactive calcium and phosphate.

Important:
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are retained in the mouth, the more effective the result.

Applied paste containing bio-available calcium and phosphate

1. Replace cap after use.
2. Helps neutralize acid challenges from acidogenic bacteria in plaque.
3. Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS
RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE INDICATIONS

- a. Prior to and following in-office bleaching.
- b. After ultrasonic, hand scaling or root planing.
- c. Following periodontal surgery (P.T.C.).
- d. After application of topical fluoride.
- e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
- f. During orthodontic treatment.
- g. For high risk caries patients.
- h. For special needs patients.

In-office application:

1. Custom Tray Application

1. Before use rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of GC Tooth Mousse into the tray and apply to upper and/or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to retain for as long as possible (1- 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual GC Tooth Mousse in the tray should be rinsed or brushed off under running water immediately after use.

II. Non-tray application:

1. If the patient remains, allow any excess saliva on the tooth surface with a cotton roll or paper. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.
3. Apply a sufficient amount for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

At home application:

II. Day time application after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1- 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1- 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

FLAVOURS
Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)
Note: S, M, V, I and T in parentheses are abbreviations for each flavour. These abbreviations are marked on the tubes after lot numbers to show the flavour of the contents.
For example, 021003S..... Strawberry

STORAGE
Store in a cool dry low humidity area away from direct sunlight (8-25°C (46-47.0°F)).
(Shelf life: 2 years from date of manufacture)

PACKAGES
Tubs containing 40g (35mL)
1. Assorted Package 10 x 40g (2 each) of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)
2. Single Flavour Package 10 x 40g in each flavour
Vanilla, Strawberry, Melon, Tutti-Frutti, Mint
3. Refill Package 1 x 40g tube in each flavour

NOTE
GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia.
*RECALDENT is a trademark used under licence. CPP-ACP technology is used by other patients or patents pending in Australia, NZ, Europe, Canada, and USA.

CAUTION
1. Always replace cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms occur, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician

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ВНИМАНИЕ
1. Always replace cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms occur, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product and refer to a physician

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